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Introduction : Dear Customer, thank you for your purchase. Please read this manual carefully before using your heated lumbar belt.

INSTRUCTIONS FOR USE

POWER ON / OFF:

1. Charge the battery for 3 to 4 hours before first use.
2. Connect the battery to the cable in the designated pocket.
3. Turn on your product by pressing and holding the heating button until the LED lights up.
4. Press the button again to select the heating level (3 levels).
5. Finally, press and hold the heating button to turn it off and disconnect the battery..

HEAT SETTINGS:

Each press of the heating button changes the heating level (1, 2, 3)

- Level 1 : Blue light on, Low heat
- Level 2 : White light on, Medium heat
- Level 3 : Red light on, High heat

Information: The G-Heat heated lumbar belt is designed to gradually reach the desired temperature over approximately ten minutes.

Tip: If your heated belt does not turn on, check that the battery is switched on and charged !

RECYCLING

WARNING : Do not dispose of battery in the trash
Please dispose of it by recycling according to local regulations.
For more information, contact your local authorities.

PRODUCT FEATURES · MAINTENANCE · WARRANTY

TECHNICAL INFORMATION: BATG01 (OPTIONAL)

Power : 7,4 V - 2 200 mAh. Up to 500 charge cycles.

Disconnect and charge the battery before storing it in a dry place at room temperature. It is strongly recommended to charge it every 2 to 3 months to extend its lifespan.

Tip: Charge the battery before and after each use.

Frequent insufficient training over a period of several weeks can lead to reduced performance.

WASHING INSTRUCTIONS:

Surface cleaning with a damp cloth is recommended. Do not machine wash. Do not tumble dry, bleach, or dry clean. Do not wring the product to remove water. Do not iron. Remove the battery and protect the connectors beforehand. Dry the connectors before reconnecting the battery.

More information, visit <https://www.g-heat.eu/pages/batteries-and-textiles-care>

WARRANTY:

Refer to the Terms and Conditions : <https://www.g-heat.eu/pages/terms-and-conditions>

BREAKDOWN

Visit <https://www.g-heat.com/pages/notices> or contact customer support at <https://www.g-heat.com/pages/nous-contacter/contact-us> or support@g-heat.com

USEFUL INFORMATION

WARNING :

If you suffer from health or heart conditions, consult your doctor before use.
Do not leave the battery charging unattended.
Unplug the battery once fully charged to optimize its lifespan.
Do not use any battery other than the one provided (BATG01).
Charge only with a 220V 2A / 5V max power adapter.
Using another charger may damage the battery.
Do not expose the battery charger to moisture.
Do not expose to temperatures below -20°C or above 60°C.
Disconnect the battery if the lumbar belt is not in use.
Do not use if any electrical components are damaged.

MORE INFORMATION

For more information, scan the QR code below with your smartphone:

SCAN ME

WWW.G-HEAT.COM

G-Heat® is a brand of G-Heat Group SAS with a share capital of €5000, registered with the Nanterre Trade and Companies Register under identification number 830 579 637. All rights reserved.

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